

ARK™ Levetiracetam II Calibrator

This ARK Diagnostics, Inc. package insert for the ARK Levetiracetam II Calibrator must be read carefully prior to use. Package insert instructions must be followed accordingly. Reliability of the assay results cannot be guaranteed if there are any deviations from the instructions in this package insert.

CUSTOMER SERVICE

 ARK Diagnostics, Inc.

48089 Fremont Blvd
Fremont, CA 94538 USA
Tel: 1-877-869-2320
Fax: 1-510-270-6298





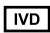



customersupport@ark-tdm.com

www.ark-tdm.com



Emergo Europe
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands

KEY TO SYMBOLS USED

	Batch code	 YYYY-MM-DD	Use by/Expiration date
	Catalog Number		Manufacturer
	<i>In Vitro</i> Diagnostic Medical Device		Temperature limitation
	Consult Instructions for Use		Calibrator
Rx Only	For Prescription Use Only		

1 NAME

ARK™ Levetiracetam II Calibrator

2 INTENDED USE

ARK Levetiracetam II Calibrator is intended for use in calibration of the ARK Levetiracetam II Assay.

3 CONTENT

ARK Levetiracetam II Calibrator is comprised of a synthetic protein matrix with the following concentrations of levetiracetam:

REF	Product Description	Quantity/Volume
5070-0002-00	ARK Levetiracetam II Calibrators Levetiracetam, buffer, bovine serum albumin, and preservatives	Dropper Vials
	A	0.0 µg/mL
	B	5.0 µg/mL
	C	12.5 µg/mL
	D	25.0 µg/mL
	E	50.0 µg/mL
	F	100.0 µg/mL

4 STANDARDIZATION

There is no internationally recognized standard for levetiracetam. ARK Levetiracetam II Calibrators are prepared by gravimetric dilution of high purity levetiracetam into a synthetic proteinaceous matrix free of levetiracetam.

5 WARNINGS AND PRECAUTIONS

- For *In Vitro* Diagnostic Use.
- Do not mix calibrators from different lot numbers.
- Use each lot as a set.
- Calibrators contain ≤0.09% sodium azide.

6 INSTRUCTIONS FOR USE

- For a complete summary and explanation of the Levetiracetam II Assay, refer to the package insert for the ARK Levetiracetam II Assay, [REF](#) 5070-0001-00 and 5070-0001-01.
- Calibrators are ready to use. Mix each level by gentle inversion before dispensing.
- Squeeze sufficient volume (~40µL/drop) into individual sample cups for each level. Consult instrument-specific sample volume requirements. Return caps to their original containers and keep tight.
- Store at 2-8°C. Use prior to the expiration date.

7 PROCEDURE

Calibration

Perform a full calibration (6- point) procedure; test calibrators in duplicate. Verify the calibration curve with at least two levels of quality controls according to the established laboratory quality assurance plan.

When to Re-Calibrate

- Whenever a new lot number of reagents is used
- Whenever indicated by quality control results
- Whenever required by standard laboratory protocols

8 LIMITATIONS OF PROCEDURE

Accurate and reproducible results are dependent upon properly functioning instruments, reagents, calibrators, controls, storage of product as directed, and good laboratory technique.

9 TRADEMARKS

ARK™ is a trademark of ARK Diagnostics, Inc.

Other brand or product names are trademarks of their respective holders.